

# *Kodak Point of Care CR systems*

## DICOM Conformance Statement

Kodak QC

Software Version 3.0

July 16, 2008

Document #DD000092

Revision 2.1

## Table of Contents

|                                                       |           |
|-------------------------------------------------------|-----------|
| <b>ACRONYMS, ABBREVIATIONS AND SYMBOLS</b>            | <b>5</b>  |
| <b>1. INTRODUCTION</b>                                | <b>6</b>  |
| 1.1. Overview                                         | 6         |
| 1.2. References                                       | 6         |
| 1.3. Definitions                                      | 6         |
| <b>2. IMPLEMENTATION MODEL</b>                        | <b>8</b>  |
| 2.1. Image Transfer                                   | 8         |
| 2.1.1. Application Data Flow                          | 9         |
| 2.1.2. Functional Definitions of Application Entities | 9         |
| 2.1.3. Sequencing of Real-World Activities            | 9         |
| 2.2. Storage Commitment                               | 9         |
| 2.2.1. Application Data Flow                          | 10        |
| 2.2.2. Functional Definitions of Application Entities | 10        |
| 2.2.3. Sequencing of Real-World Activities            | 10        |
| 2.3. Modality Worklist                                | 11        |
| 2.3.1. Application Data Flow                          | 11        |
| 2.3.2. Functional Definitions of Application Entities | 11        |
| 2.3.3. Sequencing of Real-World Activities            | 11        |
| 2.4. Print                                            | 11        |
| 2.4.1. Application Data Flow                          | 12        |
| 2.4.2. Functional Definitions of Application Entities | 12        |
| 2.4.3. Sequencing of Real-World Activities            | 12        |
| 2.5. MEDIA Storage                                    | 13        |
| 2.5.1. Application Data Flow                          | 13        |
| <b>3. KODAK QC IMAGE TRANSFER AE SPECIFICATION</b>    | <b>14</b> |
| 3.1. Association Establishment Policy                 | 14        |
| 3.1.1. General                                        | 14        |
| 3.1.2. Number of Associations                         | 14        |
| 3.1.3. Asynchronous nature                            | 14        |
| 3.1.4. Implementation Identifying Information         | 14        |
| 3.2. Association Initiation by Real World Activity    | 14        |
| 3.2.1. Export Image Request                           | 14        |
| 3.3. Association Acceptance Policy                    | 15        |

|           |                                                     |           |
|-----------|-----------------------------------------------------|-----------|
| <b>4.</b> | <b>KODAK QC STORAGE COMMITMENT AE SPECIFICATION</b> | <b>16</b> |
| 4.1.      | Association Establishment Policy                    | 16        |
| 4.1.1.    | General                                             | 16        |
| 4.1.2.    | Number of Associations                              | 16        |
| 4.1.3.    | Asynchronous nature                                 | 16        |
| 4.1.4.    | Implementation Identifying Information              | 16        |
| 4.2.      | Association Initiation by Real World Activity       | 16        |
| 4.2.1.    | Storage commitment Request                          | 16        |
| 4.3.      | Association Acceptance Policy                       | 17        |
| <b>5.</b> | <b>KODAK QC MWL AE SPECIFICATION</b>                | <b>17</b> |
| 5.1.      | Association Establishment Policy                    | 17        |
| 5.1.1.    | General                                             | 17        |
| 5.1.2.    | Number of Associations                              | 17        |
| 5.1.3.    | Asynchronous nature                                 | 17        |
| 5.1.4.    | Implementation Identifying Information              | 17        |
| 5.2.      | Association Initiation by Real World Activity       | 17        |
| 5.2.1.    | Modality Worklist Retrieval Request                 | 17        |
| 5.3.      | Association Acceptance Policy                       | 18        |
| <b>6.</b> | <b>KODAK QC PRINT AE SPECIFICATION</b>              | <b>19</b> |
| 6.1.      | Association Establishment Policy                    | 19        |
| 6.1.1.    | General                                             | 19        |
| 6.1.2.    | Number of Associations                              | 19        |
| 6.1.3.    | Asynchronous nature                                 | 19        |
| 6.1.4.    | Implementation Identifying Information              | 19        |
| 6.2.      | Association Initiation by Real World Activity       | 19        |
| 6.2.1.    | Print Request                                       | 19        |
| 6.3.      | Association Acceptance Policy                       | 21        |
| <b>7.</b> | <b>MEDIA STORAGE AE</b>                             | <b>22</b> |
| 7.1.1.    | Media Storage Application Profiles                  | 22        |
| <b>8.</b> | <b>COMMUNICATIONS PROFILES</b>                      | <b>22</b> |
| 8.1.      | Supported Communication Stacks                      | 22        |
| 8.2.      | OSI Stack                                           | 22        |
| 8.3.      | TCP/IP Stack                                        | 22        |
| 8.3.1.    | API                                                 | 22        |
| 8.3.2.    | Physical Media Support                              | 22        |
| 8.4.      | Point to Point Stack                                | 22        |
| <b>9.</b> | <b>CONFIGURATION</b>                                | <b>23</b> |
| 9.1.      | KODAK QC IMAGE TRANSFER AE Configuration            | 23        |

|         |                                                         |    |
|---------|---------------------------------------------------------|----|
| 9.2.    | KODAK QC STORAGE COMMIMENT AE Configuration             | 23 |
| 9.3.    | KODAK QC MWL AE Configuration                           | 23 |
| 9.4.    | KODAK QC PRINT AE Configuration                         | 23 |
| 10.     | SUPPORT OF EXTENDED CHARACTER SETS                      | 23 |
| 11.     | ANNEX A - INFORMATION OBJECT DEFINITIONS                | 24 |
| 11.1.   | Computed Radiography (CR) Information Object Definition | 24 |
| 11.1.1. | Information Object Definitions For CR Image IOD         | 24 |

## Acronyms, Abbreviations and Symbols

|                |                                                    |
|----------------|----------------------------------------------------|
| ACC            | American College of Cardiology                     |
| ACR            | American College of Radiology                      |
| ASCII          | American Standard Code for Information Interchange |
| AE             | Application Entity                                 |
| ANSI           | American National Standards Institute              |
| CD             | Compact Disc                                       |
| CR             | Computed Radiography                               |
| DICOM          | Digital Imaging and COmmunications in Medicine     |
| DIMSE          | DICOM Message Service Element                      |
| DIMSE-C        | DICOM Message Service Element-Composite            |
| DIMSE-N        | DICOM Message Service Element-Normalized           |
| FSC            | File Set Creator                                   |
| HD             | Hard Disc                                          |
| HIS            | Hospital Information System                        |
| HL7            | Health Level 7                                     |
| IE             | Information Entity                                 |
| IOD            | Information Object Definition                      |
| ISO            | International Standard Organization                |
| MOD            | Magneto Optical Disk                               |
| MWL            | Modality Worklist                                  |
| MWM            | Modality Worklist Management                       |
| NEMA           | National Electrical Manufacturers Association      |
| PDU            | Protocol Data Unit                                 |
| RIS            | Radiology Information System                       |
| SC             | Secondary Capture                                  |
| STORAGE COMMIT | Storage Commitment                                 |
| SCP            | Service Class Provider                             |
| SCU            | Service Class User                                 |
| SOP            | Service Object Pair                                |
| TCP/IP         | Transmission Control Protocol/Internet Protocol    |
| UID            | Unique Identifier                                  |

# 1. Introduction

This document is a DICOM Conformance Statement for the Kodak QC system.

This document is intended to provide the reader with the knowledge of how to integrate this product within a DICOM compliant hospital network. It details DICOM Service Classes and Communication Protocols that are supported in Kodak QC.

Kodak QC supports following service classes:

- Storage Service Class (SCU)
- Storage Commitment Service Class (SCU)
- Modality Worklist Management Service Class (SCU)
- Print Service Class (SCU)
- Media Storage Service (FSC)

It is recommended that the reader be familiar with the DICOM Standard (referenced in section 1.2) prior to reading this DICOM Conformance Statement.

## 1.1. Overview

This document contains 10 sections (including this Section 1 – Introduction).

Section 2 presents the DICOM implementation model of the Kodak QC, supported by detailed application data flow diagrams.

Section 3 provides definition of the Image Transfer Application Entity (AE) that is part of Kodak QC. Details are given in the AE Specification.

Section 4 provides definition of the Modality Worklist AE that is part of Kodak QC. Details are given in the AE Specification.

Communication Profiles are discussed in section 7, and configuration required by Kodak QC is provided in Section 8. Section 9 indicates support for Extended Character Sets.

Annex A provides Information Object Definitions for the modalities supported by Kodak QC.

## 1.2. References

ACR-NEMA Digital Imaging and Communications in Medicine, DICOM V3.0,.

## 1.3. Definitions

- **Association Establishment** - An Association Establishment is the first phase of communication between two DICOM Application Entities (AEs). The AEs use the Association Establishment to negotiate how data will be encoded and the type of data to be exchanged.
- **Called Application Entity Title** - The Called AE Title defines the intended receiver of an association request.

*Kodak Point of Care CR Systems, DICOM Conformance Statement*

- **Calling Application Entity Title** - The Calling AE Title defines the requestor of an association request.
- **DICOM Message Service Element (DIMSE)** - A DIMSE defines the services and protocols utilized by an Application Entity to exchange messages.
- **Information Object Definition (IOD)** - An IOD is a data model, which is an abstraction of real-world information. This data model defines the nature and attributes relevant to the class of real-world objects represented.
- **Service Class Provider (SCP)** - A Service Class Provider plays the "server" role to perform operations and invoke notifications during an Association. An example of a Storage Service Class Provider would be an image storage device. In this case, the image storage device is storing the image that was sent by a Service Class User.
- **Service Class User (SCU)** - A Service Class User plays the "client" role to invoke operations and perform notifications during an Association. An example of a Storage Service Class User would be an image acquisition device. In this case, the image acquisition device will create and send a DICOM image by requesting that a Service Class Provider store that image.
- **Service/Object Pair (SOP) Class** - A SOP Class is defined by the union of an Information Object Definition and a set of DIMSE Services. A DICOM Application Entity may support one or more SOP Classes. Each SOP Class is uniquely identified by a SOP Class UID.
- **SOP Instance** - A specific occurrence of an Information Object.
- **Transfer Syntax** - The Transfer Syntax is a set of encoding rules that allow DICOM Application Entities to negotiate the encoding techniques (e.g. data element structure, byte ordering, compression) they are able to support. The Transfer Syntax is negotiated during Association Negotiation.
- **Unique Identifier (UID)** - A Unique Identifier is a globally unique, ISO compliant, ASCII-numeric string. It guarantees uniqueness across multiple countries, sites, vendors and equipment.
- **DICOM File** - A DICOM File is a file formatted in accordance with Part 10 of the DICOM Standard.

## 2. Implementation Model

Kodak QC is a medical acquisition system that supports Computed Radiography (CR).

Kodak QC encompasses the following DICOM Application Entities:

- KODAK QC IMAGE TRANSFER AE - to exchange images with other Application Entities by the means of DICOM network exchange. It implements the following Service Class:
  - Storage as SCU
- KODAK QC STORAGE COMMITMENT AE - to ask for safe storage confirmation of images that were already sent to other Application Entities in previous associations by the means of DICOM network exchange. It implements the following Service Class:
  - Storage Commitment as SCU
- KODAK QC MWL AE - to fetch modality worklist information from a worklist manager. It implements the following Service Class:
  - DICOM Modality Worklist SCU
- KODAK QC PRINT AE - to exchange print pages with other Application Entities by the means of DICOM network exchange. It implements the following Service Class:
  - Print as SCU

### 2.1. Image Transfer

KODAK QC IMAGE TRANSFER AE is implemented as a single application entity and performs the following services:

- sends/exports images to remote DICOM storage devices (acts as Storage SCU)



### 2.1.1. Application Data Flow

Figure 1 illustrates the following scenarios:

- Send a **STORE Request** to a remote DICOM AE in case of automatic or manual sending request.

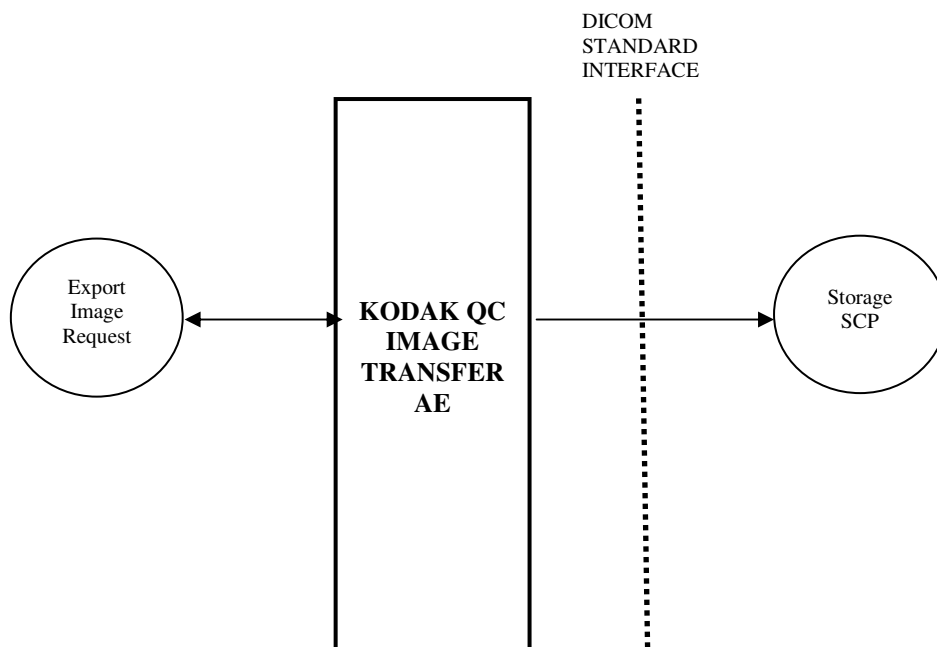


Figure 1: Image Transfer Data Flow Diagram

### 2.1.2. Functional Definitions of Application Entities

The KODAK QC IMAGE TRANSFER AE component operates as a service. The startup sequence of the Kodak QC system initiates its execution. The service is NOT shut down when the Kodak QC application is closed

### 2.1.3. Sequencing of Real-World Activities

- Automatic sending of images - When an Image is accepted in the scan mode and a default DICOM store SCP (i.e. PACS, etc.) is configured, then the image is being sent to it.
- Manual sending of images - the user can enter the Archive tool from the user interface and select images to be sent to a specific storage device.

## 2.2. Storage Commitment

KODAK QC Storage Commitment AE is implemented as a single application entity and performs the following services:

- sends/exports N\_ACTION requests to remote DICOM Image Manager for previously stored images (acts as Storage Commitment SCU).

*Kodak Point of Care CR Systems, DICOM Conformance Statement*

- receives N\_EVENT\_REPORT requests from DICOM Image manager on a separate association initiated by the DICOM Image Manager.

### 2.2.1. Application Data Flow

Figure 2 illustrates the following scenarios:

- Send an **N\_ACTION request** to a remote DICOM AE once the preceding image transfer request is successful and associated to the storage commitment destination.

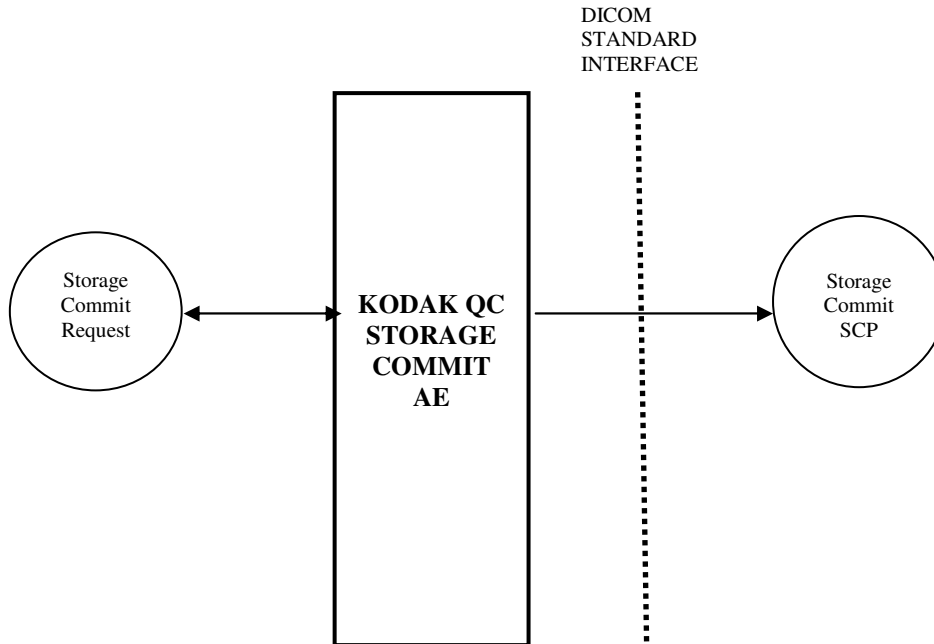


Figure 2: Storage Commitment Data Flow Diagram

### 2.2.2. Functional Definitions of Application Entities

The KODAK QC STORAGE COMMIT AE component operates as a service. The startup sequence of the Kodak QC system initiates its execution. The service is NOT shut down when the Kodak QC application is closed

### 2.2.3. Sequencing of Real-World Activities

- After a successful transfer of image (as described at 2.1.3 ) to a device with Store Commit SCP capability then the AE initiates a new association for sending N\_ACTION request using the previous image SOP Class/Instance UID pair.

- The service always listens (waits for association, on local port 105) for a different association initiated by the Storage Commit SCP to receive the response of the commitment requests (N\_EVENT\_REPORT) and updates its as Success/Failure in the Kodak QC database.

## 2.3. Modality Worklist

### 2.3.1. Application Data Flow

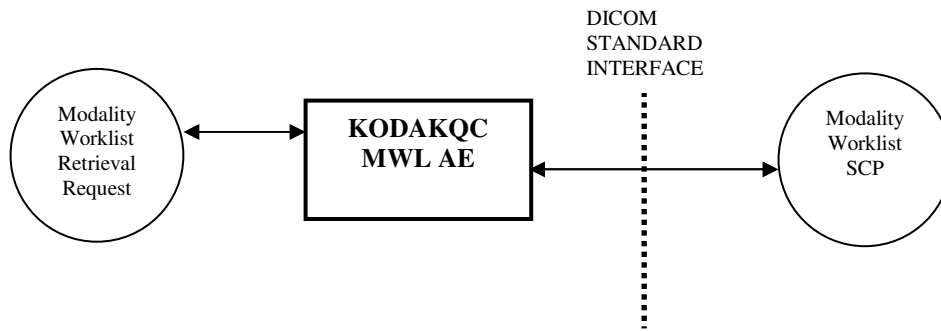


Figure 3: Modality Worklist Data Flow Diagram

### 2.3.2. Functional Definitions of Application Entities

KODAK QC MWL AE is used to query modality worklist information from a remote DICOM device. It therefore performs the following tasks:

- Establishes a DICOM Association with a remote DICOM device.
- Issues a request for a worklist to a remote DICOM device.
- Receives Modality Worklist from a remote DICOM device.

KODAK QC MWL AE component operates as part of the Kodak QC application. It is shut down when the application terminates.

### 2.3.3. Sequencing of Real-World Activities

Not applicable.

## 2.4. Print

KODAK QC PRINT AE is implemented as a single application entity and performs the following services:

- creates print film sessions, film boxes and image boxes with remote DICOM Print device as laser imagers etc. (acts as Print SCU)
- sends presentation LUT (IDENTITY + LIN OD) to Print device.
- receives N\_EVENT\_REPORT for Print Job from print device with status of print request (written in the **Event Viewer** of the computer ).

### 2.4.1. Application Data Flow

Figure 4 illustrates the following scenarios:

- Send a **PRINT Request** to a remote DICOM AE in case of automatic or manual printing request.

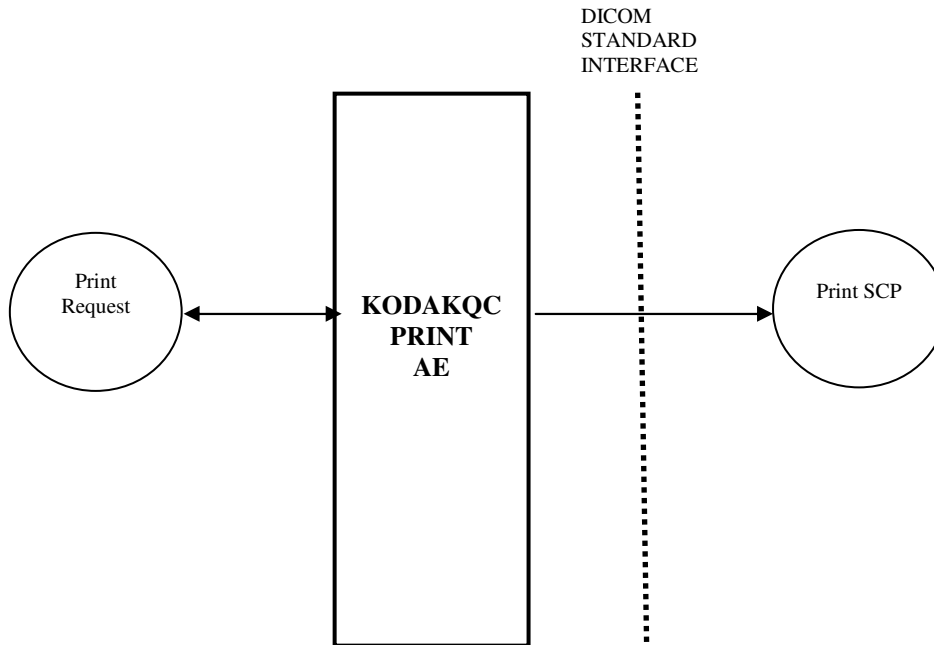


Figure 4: Print Data Flow Diagram

### 2.4.2. Functional Definitions of Application Entities

KODAK QC PRINT AE acts as a Windows Printer Driver. The AE installation and configuration has a dedicated “Add DICOM Printer Wizard”.

### 2.4.3. Sequencing of Real-World Activities

- Automatic sending of single image print page - When an Image is accepted in the scan mode and a default DICOM Print SCP is configured, and then the print request of the image is sent.

- Manual printing of images - the user can enter the Viewer tool of the user interface and select images to be printed.

## 2.5. MEDIA Storage

KODAK QC MEDIA Storage AE is implemented as a single application entity and performs the following services:

- Burns Images and DICOMDIR on media. (acts as FSC)

### 2.5.1. Application Data Flow

Figure 5 illustrates the following scenarios:

- Request a **BURN operation** .

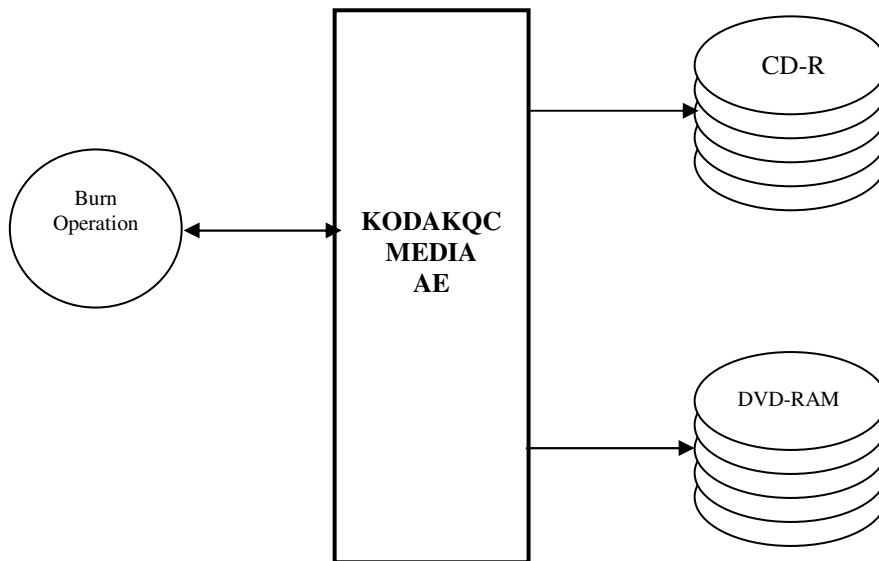


Figure 5: Burn Media Flow Diagram

### 3. KODAK QC IMAGE TRANSFER AE Specification

KODAK QC IMAGE TRANSFER AE provides Standard Conformance to the following DICOM SOP Class as an SCU:

| SOP Class Name   | SOP Class UID             |
|------------------|---------------------------|
| CR Image Storage | 1.2.840.10008.5.1.4.1.1.1 |

#### 3.1. Association Establishment Policy

##### 3.1.1. General

The maximum length of PDU that the KODAK QC IMAGE TRANSFER AE will receive is 16384.

##### 3.1.2. Number of Associations

The KODAK QC IMAGE TRANSFER AE can initiate one association concurrently.

##### 3.1.3. Asynchronous nature

The KODAK QC IMAGE TRANSFER AE allows a single outstanding operation on any association. Therefore, Kodak QC AE does not support asynchronous operations and will not perform asynchronous window negotiation, other than the default as specified by the DICOM specification.

##### 3.1.4. Implementation Identifying Information

The KODAKQC IMAGE TRANSFER AE is identified by the following ids:

Implementation Class UID: 1.2.3

Implementation Version Name: ver 1

#### 3.2. Association Initiation by Real World Activity

This section details the action of the KODAKQC IMAGE TRANSFER AE SCU component as a result of user initiated activity on the Kodak QC Application User Interface.

##### 3.2.1. Export Image Request

###### 3.2.1.1. Associated Real World Activity

Automatic export:

The user configures the Kodak QC application to automatically export a study after it has been completed. The user sets up the default destination and no further intervention on the user part is required. The KODAK QC IMAGE TRANSFER AE transfers the images in the background.

### 3.2.1.2. Proposed Presentation Contexts

| Presentation Context Table |                           |                           |                   |      |                      |
|----------------------------|---------------------------|---------------------------|-------------------|------|----------------------|
| Abstract Syntax            |                           | Transfer Syntax           |                   | Role | Extended Negotiation |
| Name                       | UID                       | Name List                 | UID List          |      |                      |
| CR Image Storage           | 1.2.840.10008.5.1.4.1.1.1 | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCU  | None                 |

### 3.2.1.3. SOP-Specific Conformance

This implementation supports transfers as an SCU as described in the DICOM Standard, PS3.4, Annex B.

The Information Object attributes that are encapsulated in a dataset and sent with the C-STORE request are specified in Annex A of this document.

## 3.3. Association Acceptance Policy

NA.

## 4. KODAK QC STORAGE COMMITMENT AE Specification

KODAK QC STORAGE COMMIT AE provides Standard Conformance to the following DICOM SOP Class as an SCU:

| SOP Class Name                      | SOP Class UID        |
|-------------------------------------|----------------------|
| Storage Commitment Push Model Class | 1.2.840.10008.1.20.1 |

### 4.1. Association Establishment Policy

#### 4.1.1. General

The maximum length of PDU that the KODAK QC STORAGE COMMIT AE will receive is 16384.

#### 4.1.2. Number of Associations

The KODAK QC STORAGE COMMIT AE can initiate one association concurrently..

#### 4.1.3. Asynchronous nature

The KODAK QC STORAGE COMMIT AE allows a single outstanding operation on any association. Therefore, Kodak QC AE does not support asynchronous operations and will not perform asynchronous window negotiation, other than the default as specified by the DICOM specification.

#### 4.1.4. Implementation Identifying Information

The KODAK QC STORAGE COMMIT AE is identified by the following ids:

Implementation Class UID: 1.2.3

Implementation Version Name: ver 1

### 4.2. Association Initiation by Real World Activity

This section details the action of the KODAK QC STORAGE COMMIT AE SCU component as a result of user initiated activity on the Kodak QC Application User Interface.

#### 4.2.1. Storage commitment Request

##### 4.2.1.1. Associated Real World Activity

Automatic N\_ACTION requests after sending images to storage device

##### 4.2.1.2. Proposed Presentation Contexts

| Presentation Context Table          |                      |                           |                   |      |                      |
|-------------------------------------|----------------------|---------------------------|-------------------|------|----------------------|
| Abstract Syntax                     |                      | Transfer Syntax           |                   | Role | Extended Negotiation |
| Name                                | UID                  | Name List                 | UID List          |      |                      |
| Storage Commitment Push Model Class | 1.2.840.10008.1.20.1 | Implicit VR Little Endian | 1.2.840.10008.1.2 | SCU  | None                 |

##### 4.2.1.3. SOP-Specific Conformance

This implementation supports storage commitment requests as an SCU as described in the DICOM Standard, PS3.4, Annex B.

*Kodak Point of Care* CR Systems, DICOM Conformance Statement



### 4.3. Association Acceptance Policy

The KODAK QC STORAGE COMMIT AE accepts Associations on port 105 for N\_EVENT\_REPORT requests as an SCU, and C\_ECHO requests as an SCP.

## 5. KODAK QC MWL AE Specification

### 5.1. Association Establishment Policy

#### 5.1.1. General

When KodakQC Application issues a request to retrieve a Modality Worklist, it initiates an association to the Modality Worklist SCP through the KODAK QC MWL AE Component.

#### 5.1.2. Number of Associations

KODAK QC MWL AE initiates one association with the default remote Modality Worklist SCP. The association is released once the worklist has been retrieved.

#### 5.1.3. Asynchronous nature

The system does not support asynchronous operations and will not perform asynchronous window negotiation.

#### 5.1.4. Implementation Identifying Information

The KODAK QC MWL AE is identified by the following ids:

Implementation Class UID: 1.2.840.113564.3.2

Implementation Version Name: not specified

### 5.2. Association Initiation by Real World Activity

#### 5.2.1. Modality Worklist Retrieval Request

##### 5.2.1.1. Associated Real World Activity

The user of the KodakQC Application selects the Retrieve Worklist operation button on the user interface.

##### 5.2.1.2. Proposed Presentation Contexts

| Presentation Context Table      |                        |                               |                   |      |                      |
|---------------------------------|------------------------|-------------------------------|-------------------|------|----------------------|
| Abstract Syntax                 |                        | Transfer Syntax               |                   | Role | Extended Negotiation |
| Name                            | UID                    | Name                          | UID               |      |                      |
| Modality Worklist Model<br>FIND | 1.2.840.10008.5.1.4.31 | Implicit VR,<br>Little Endian | 1.2.840.10008.1.2 | SCU  | None                 |

### 5.2.1.3. SOP-Specific Conformance for Modality Worklist Model- Find SOP Class

| Attribute                             | Tag         | Possible Values                                                                                                                                                            |
|---------------------------------------|-------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Specific Character Set                | (0008,0005) | User Defined or Empty                                                                                                                                                      |
| Accession Number                      | (0008,0050) | User Defined or default (* / Empty)                                                                                                                                        |
| Referring Physician's Name            | (0008,0090) | Empty                                                                                                                                                                      |
| Patient's Name                        | (0010,0010) | User Defined or default (* / Empty)                                                                                                                                        |
| Patient ID                            | (0010,0020) | User Defined or default (* / Empty)                                                                                                                                        |
| Patient's Birth Date                  | (0010,0030) | Empty                                                                                                                                                                      |
| Patient's Sex                         | (0010,0040) | Empty                                                                                                                                                                      |
| Study Instance UID                    | (0020,000D) | Empty                                                                                                                                                                      |
| Requested Procedure Description       | (0032,1060) | Empty                                                                                                                                                                      |
| Scheduled Procedure Step Sequence     | (0040,0100) | CR or Empty<br>configurable in DICOM settings<br>Days range is configurable in DICOM settings<br>Empty<br>Empty<br>configurable in DICOM settings<br>User Defined or Empty |
| >Modality                             | (0008,0060) |                                                                                                                                                                            |
| >Scheduled Station AE Title           | (0040,0001) |                                                                                                                                                                            |
| >Scheduled Procedure Step Start Date  | (0040,0002) |                                                                                                                                                                            |
| > Scheduled Procedure Step Start Time | (0040,0003) |                                                                                                                                                                            |
| >Scheduled Procedure Step Description | (0040,0007) |                                                                                                                                                                            |
| > Scheduled Station Name              | (0040,0010) |                                                                                                                                                                            |
| >Scheduled Procedure Step Location    | (0040,0011) |                                                                                                                                                                            |

### 5.3.Association Acceptance Policy

The KODAK QC MWL AE does not accept association requests.

## 6. KODAK QC PRINT AE Specification

KODAK QC PRINT AE provides Standard Conformance to the following DICOM SOP Class as an SCU:

| SOP Class Name                                     | SOP Class UID           |
|----------------------------------------------------|-------------------------|
| Basic Grayscale Print Management<br>Meta SOP Class | 1.2.840.10008.5.1.1.9   |
| Basic Film Session SOP Class                       | 1.2.840.100008.5.1.1.1  |
| Basic Film Box SOP Class                           | 1.2.840.100008.5.1.1.2  |
| Basic Grayscale Image Box SOP Class                | 1.2.840.100008.5.1.1.4  |
| Printer SOP Class                                  | 1.2.840.100008.5.1.1.16 |
| Print Job SOP Class                                | 1.2.840.10008.5.1.1.14  |
| Presentation LUT SOP Class                         | 1.2.840.10008.5.1.1.23  |

### 6.1. Association Establishment Policy

#### 6.1.1. General

The maximum length of PDU that the KODAK QC PRINT AE will receive is 16384.

#### 6.1.2. Number of Associations

The KODAK QC PRINT AE can initiate one association concurrently.

#### 6.1.3. Asynchronous nature

The KODAK QC PRINT AE allows a single outstanding operation on any association. Therefore, KodakQC AE does not support asynchronous operations and will not perform asynchronous window negotiation, other than the default as specified by the DICOM specification.

#### 6.1.4. Implementation Identifying Information

The KODAK QC PRINT AE is identified by the following ids:

Implementation Class UID: 1.2.840.113564.12.1

Implementation Version Name: not specified

### 6.2. Association Initiation by Real World Activity

This section details the action of the KODAK QC PRINT AE SCU component as a result of user initiated activity on the KodakQC Application User Interface.

#### 6.2.1. Print Request

##### 6.2.1.1. Associated Real World Activity

Automatic or manual Print requests after images are scanned

### 6.2.1.2. Proposed Presentation Contexts

| Presentation Context Table                            |                        |                              |                   |      |                      |
|-------------------------------------------------------|------------------------|------------------------------|-------------------|------|----------------------|
| Abstract Syntax                                       |                        | Transfer Syntax              |                   | Role | Extended Negotiation |
| Name                                                  | UID                    | Name List                    | UID List          |      |                      |
| Basic Grayscale<br>Print Management<br>Meta SOP Class | 1.2.840.10008.5.1.1.9  | Implicit VR<br>Little Endian | 1.2.840.10008.1.2 | SCU  | None                 |
| Print Job SOP Class                                   | 1.2.840.10008.5.1.1.14 | Implicit VR<br>Little Endian | 1.2.840.10008.1.2 | SCU  | None                 |
| Presentation LUT<br>SOP Class                         | 1.2.840.10008.5.1.1.23 | Implicit VR<br>Little Endian | 1.2.840.10008.1.2 | SCU  | None                 |

### 6.2.1.3. SOP-Specific Conformance for Basic Film Session SOP Class

| Attribute        | SCU Usage | Tag         | Possible Values                                     | Notes                                         |
|------------------|-----------|-------------|-----------------------------------------------------|-----------------------------------------------|
| Number of Copies | U         | (2000,0010) |                                                     |                                               |
| Print Priority   | U         | (2000,0020) | HIGH, MED, LOW                                      | print_priority<br>ExtEscape<br>command exists |
| Medium Type      | U         | (2000,0030) | PAPER, CLEAR<br>FILM, BLUE FILM,<br>MAMMO BLUE FILM |                                               |
| Film Destination | U         | (2000,0040) | BIN_n                                               | n is printer<br>specific                      |

### 6.2.1.4. SOP-Specific Conformance for Basic Film Box SOP Class

| Attribute                  | SCU Usage | Tag         | Possible Values               | Notes                            |
|----------------------------|-----------|-------------|-------------------------------|----------------------------------|
| Image Display<br>Format    | M         | (2010,0010) | STANDARD/1,1                  | 1-up only                        |
| Film Orientation           | M         | (2010,0040) | PORTRAIT,<br>LANDSCAPE        |                                  |
| Film Size ID               | M         | (2010,0050) |                               | Printer specific                 |
| Magnification              | U         | (2010,0060) | CUBIC, BILINEAR,<br>REPLICATE |                                  |
| Type                       |           |             |                               |                                  |
| Max Density                | U         | (2010,0130) | 1 - 399                       | dmax ExtEscape<br>command exists |
| Border Density             | U         | (2010,0100) | BLACK, WHITE                  |                                  |
| Empty Image<br>Density     | U         | (2010,0110) | Same as Border Density        |                                  |
| Min Density                | U         | (2010,0120) | 1 - 399                       |                                  |
| Illumination               | MC        | (2010,015E) | 0 - 3000                      | Default to 2000                  |
| Reflected Ambient<br>Light | MC        | (2010,0160) | 0 - 200                       | Default to 10                    |

### 6.2.1.5. SOP-Specific Conformance for Basic Image Box SOP Class

| Attribute                         | SCU Usage | Tag         | Possible Values               | Notes                           |
|-----------------------------------|-----------|-------------|-------------------------------|---------------------------------|
| Image Position                    | M         | (2020,0010) |                               |                                 |
| Basic Grayscale<br>Image Sequence | M         | (2020,0110) |                               |                                 |
| >Samples per Pixel                | M         | (0028,0002) | 1                             |                                 |
| >Photometric<br>Interpretation    | M         | (0028,0004) | MONOCHROME1,<br>MONOCHROME2   |                                 |
| >Rows                             | M         | (0028,0010) |                               |                                 |
| >Columns                          | M         | (0028,0011) |                               |                                 |
| >Pixel Aspect<br>Ratio            | MC        | (0028,0034) |                               |                                 |
| >Bits Allocated                   | M         | (0028,0100) | 8, 16                         |                                 |
| >Bits Stored                      | M         | (0028,0101) | Up to 12                      | bpp ExtEscape<br>command exists |
| >High Bit                         | M         | (0028,0102) | Bits Stored - 1               |                                 |
| >Pixel<br>Representation          | M         | (0028,0103) | 0000H                         |                                 |
| >Pixel Data                       | M         | (7FE0,0010) |                               |                                 |
| >Polarity                         | U         | (2020,0020) | NORMAL, REVERSE               |                                 |
| >Magnification<br>Type            | U         | (2010,0060) | CUBIC, BILINEAR,<br>REPLICATE |                                 |

### 6.3. Association Acceptance Policy

NA.

## 7. Media Storage AE

This Application Entity provides conformance to the standard General Purpose CD-R (STD-GEN-CD ) and DVD (STD-GEN-DVD-RAM) interchange Profiles as an FSC.

### 7.1.1. Media Storage Application Profiles

In addition to the mandatory support of DICOMDIR for the Basic Directory SOP class (1.2.840.10008.1.3.10), refer to the following table for the list of composite image and stand-alone storage SOP class and the list of transfer syntaxes that are supported by this media storage interchange service respectively

| Presentation Context Table      |                           |                           |                     |      |                      |
|---------------------------------|---------------------------|---------------------------|---------------------|------|----------------------|
| Abstract Syntax                 |                           | Transfer Syntax           |                     | Role | Extended Negotiation |
| Name                            | UID                       | Name List                 | UID List            |      |                      |
| Media Storage Directory Storage | 1.2.840.10008.1.3.10      | Explicit VR Little Endian | 1.2.840.10008.1.2.1 | FSC  | None                 |
| CR Image Storage                | 1.2.840.10008.5.1.4.1.1.1 | Implicit VR Little Endian | 1.2.840.10008.1.2   | FSC  | None                 |

#### 7.1.1.1. SOP-Specific Conformance

The Information Object attributes that are encapsulated in a dataset via burn request are specified in Annex A of this document.

## 8. Communications Profiles

### 8.1. Supported Communication Stacks

TCP/IP Network Communication Support as defined in Part 8 of the DICOM Standard is supported.

### 8.2. OSI Stack

Not applicable to this product.

### 8.3. TCP/IP Stack

#### 8.3.1. API

The KODAK QC IMAGE TRANSFER AE KODAK QC MWL AE implementations use Berkeley style sockets.

#### 8.3.2. Physical Media Support

KodakQC is indifferent to the physical medium over which TCP/IP executes; it inherits the medium from the computer system upon which it executes.

### 8.4. Point to Point Stack

Not applicable to this product.

*Kodak Point of Care CR Systems, DICOM Conformance Statement*

## 9. Configuration

### 9.1. KODAK QC IMAGE TRANSFER AE Configuration

KodakQC Service Manual, supplied with the product, defines the available configuration parameters.

Application entity host names can be specified as either IP address or host name.

### 9.2. KODAK QC STORAGE COMMITMENT AE Configuration

KodakQC Service Manual, supplied with the product, defines the available configuration parameters.

Application entity host names can be specified as either remote IP address or host name. It will receive association requests only on local port 105.

### 9.3. KODAK QC MWL AE Configuration

KodakQC Service Manual, supplied with the product, defines the available configuration parameters.

Local AE title can be set.

Parameters of the remote MWL SCP can be set (i.e., AE title, host name or IP address)

### 9.4. KODAK QC PRINT AE Configuration

KodakQC Service Manual, supplied with the product, defines the available configuration parameters.

Application entity host names can be specified as either IP address or host name.

## 10. Support of Extended Character Sets

- ISO\_IR 100 (European Languages)
- GB18030 (Chinese)
- ISO 2022 IR 87 (Japanese for Kanji, full **only** Katakana, Hiragana).
- ISO 2022 IR 13\ISO 2022 IR 87 (Japanese for Kanji, **full and half** Katakana, Hiragana).
- ISO\_IR 144 (Cyrillic – Russian).
- ISO\_IR 148(Turkish).

## 11. Annex A - Information Object Definitions

The Information Object Definitions for Digital Computed Radiography (CR) image is described in this section.

### 11.1. Computed Radiography (CR) Information Object Definition

KodakQC supports sending of CR images.

#### 11.1.1. Information Object Definitions For CR Image IOD

##### 11.1.1.1. Patient Module

| Attribute Name                     | Tag          | Type | Attribute Assignment                           |
|------------------------------------|--------------|------|------------------------------------------------|
| Patient's Name                     | (0010, 0010) | 2    | From User or MWL                               |
| Patient ID                         | (0010, 0020) | 2    | From User or MWL                               |
| Patient's Birth Date               | (0010, 0030) | 2    | From User or MWL                               |
| Patient's Sex                      | (0010, 0040) | 2    | From User or MWL                               |
| Referenced Patient Sequence        | (0008,1120)  | 3    | Not set                                        |
| >Referenced SOP Class UID          | (0008,1150)  | 1C   | Required if Referenced Patient Sequence is set |
| >Referenced SOP Class Instance UID | (0008,1155)  | 1C   | Required if Referenced Patient Sequence is set |

##### 11.1.1.2. General Study Module

| Attribute Name                     | Tag          | Type | Attribute Assignment                         |
|------------------------------------|--------------|------|----------------------------------------------|
| Study Instance UID                 | (0020, 000D) | 1    | Auto generated or from MWL                   |
| Study Date                         | (0008, 0020) | 2    | Always set                                   |
| Study Time                         | (0008, 0030) | 2    | Always set                                   |
| Referring Physician's Name         | (0008, 0090) | 2    | From User or MWL                             |
| Study ID                           | (0020, 0010) | 2    | Length=0                                     |
| Accession Number                   | (0008, 0050) | 2    | From User or MWL                             |
| Study Description                  | (0008,1030)  | 3    | From User or MWL                             |
| Physician(s) of Record             | (0008,1048)  | 3    | Not set                                      |
| Name of Physician(s) Reading Study | (0008,1060)  | 3    | Not set                                      |
| Referenced Study Sequence          | (0008,1110)  | 3    | Not set                                      |
| >Referenced SOP Class UID          | (0008,1150)  | 1C   | Required if Referenced Study Sequence is set |
| >Referenced SOP Class Instance UID | (0008,1155)  | 1C   | Required if Referenced Study Sequence is set |

*Kodak Point of Care CR Systems, DICOM Conformance Statement*



|                           |             |    |                                                                        |
|---------------------------|-------------|----|------------------------------------------------------------------------|
| Procedure Code Sequence   | (0008,1032) | 3  | Not set                                                                |
| >Code Value               | (0008,0100) | 1C | Required if Procedure Code Sequence present                            |
| >Coding Scheme Designator | (0008,0102) | 1C | Required if Procedure Code Sequence present                            |
| >Coding Scheme Version    | (0008,0103) | 1C | Required if Procedure Code Sequence present and its value is available |
| >Code Meaning             | (0008,0104) | 1C | Required if Procedure Code Sequence present                            |

### 11.1.1.3. Patient Study Module

| Attribute Name                  | Tag          | Type | Attribute Assignment |
|---------------------------------|--------------|------|----------------------|
| Admitting Diagnoses Description | (0008,1080)  | 3    | Not set              |
| Patient's Age                   | (0010, 1010) | 3    | Not set              |
| Patient's Size                  | (0010,1020)  | 3    | Not set              |
| Patient's Weight                | (0010,1030)  | 3    | Not set              |
| Occupation                      | (0010,2180)  | 3    | Not set              |
| Additional Patient's History    | (0010,21B0)  | 3    | Not set              |

### 11.1.1.4. General Series Module

| Attribute Name                      | Tag          | Type | Attribute Assignment                                  |
|-------------------------------------|--------------|------|-------------------------------------------------------|
| Modality                            | (0008, 0060) | 1    | CR                                                    |
| Series Instance UID                 | (0020, 000E) | 1    | Always set                                            |
| Series Number                       | (0020, 0011) | 2    | Always set                                            |
| Laterality                          | (0020,0060)  | 2C   | Length=0                                              |
| Series Date                         | (0008,0021)  | 3    | Not set                                               |
| Series Time                         | (0008,0031)  | 3    | Not set                                               |
| Performing Physician's Name         | (0008,1050)  | 3    | Set From User                                         |
| Protocol Name                       | (0018,1030)  | 3    | Not set                                               |
| Series Description                  | (0018,103E)  | 3    | Not set                                               |
| Operator's Name                     | (0008,1070)  | 3    | Always set                                            |
| Referenced Study Component Sequence | (0008,1111)  | 3    | Not set                                               |
| >Referenced SOP Class UID           | (0008,1150)  | 1C   | Set if Referenced Study Component Sequence is present |

*Kodak Point of Care CR Systems, DICOM Conformance Statement*

|                                       |             |    |                                                                                       |
|---------------------------------------|-------------|----|---------------------------------------------------------------------------------------|
| >Referenced SOP Instance UID          | (0008,1155) | 1C | Set if Referenced Study Component Sequence is present                                 |
| Body Part Examined                    | (0018,0015) | 2  | Always set                                                                            |
| Patient Position                      | (0018,5100) | 2  | Length=0                                                                              |
| Smallest Pixel Value in Series        | (0028,0108) | 3  | Not set                                                                               |
| Largest Pixel Value in Series         | (0028,0109) | 3  | Not set                                                                               |
| Request Attributes Sequence           | (0040,0275) | 3  | Not set                                                                               |
| >Requested Procedure ID               | (0040,1001) | 1C | Set if Request Attributes Sequence is present                                         |
| >Scheduled Procedure Step ID          | (0040,0009) | 1C | Set if Request Attributes Sequence is present                                         |
| >Scheduled Procedure Step Description | (0040,0007) | 1C | Set if Request Attributes Sequence is present and the value is available              |
| >Scheduled Action Item Code Sequence  | (0040,0008) | 3  | Not set                                                                               |
| >>Code Value                          | (0008,0100) | 1C | Required if Scheduled Action Item Code Sequence is present                            |
| >>Coding Scheme Designator            | (0008,0102) | 1C | Required if Scheduled Action Item Code Sequence is present                            |
| >>Coding Scheme Version               | (0008,0103) | 1C | Required if Scheduled Action Item Code Sequence is present and its value is available |
| >>Code Meaning                        | (0008,0104) | 1C | Required if Scheduled Action Item Code Sequence is present                            |
| Performed Procedure Step ID           | (0040,0253) | 3  | Not set                                                                               |
| Performed Procedure Step Start Date   | (0040,0244) | 3  | Not set                                                                               |
| Performed Procedure Step Start Time   | (0040,0245) | 3  | Not set                                                                               |
| Performed Procedure Step Description  | (0040,0254) | 3  | Not set                                                                               |
| Performed Action Item Code Sequence   | (0040,0260) | 3  | Not set                                                                               |
| >Code Value                           | (0008,0100) | 1C | Required if Performed Action Item Code Sequence is present                            |
| >Coding Scheme Designator             | (0008,0102) | 1C | Required if Performed Action Item Code Sequence is present                            |
| >Coding Scheme Version                | (0008,0103) | 1C | Required if Performed Action Item Code Sequence is present and its value is available |
| >Code Meaning                         | (0008,0104) | 1C | Required if Performed Action Item Code Sequence is present                            |

#### 11.1.1.5. CR Series Module

| Attribute Name       | Tag          | Type | Attribute Assignment |
|----------------------|--------------|------|----------------------|
| Body Part Examined   | (0018, 0015) | 2    | Always set           |
| View Position        | (0018,5101)  | 2    | Always set           |
| Filter Type          | (0018,1160)  | 3    | Not set              |
| Collimator/grid Name | (0018,1180)  | 3    | Not set              |
| Focal Spot           | (0018, 1190) | 3    | Not set              |
| Plate Type           | (0018, 1260) | 3    | Not set              |
| Phosphor Type        | (0018, 1261) | 3    | Not set              |

#### 11.1.1.6. General Equipment Module

| Attribute Name                | Tag          | Type | Attribute Assignment |
|-------------------------------|--------------|------|----------------------|
| Manufacturer                  | (0008, 0070) | 2    | Always set           |
| Institution Name              | (0008, 0080) | 3    | Always set           |
| Institution Address           | (0008, 0080) | 3    | Not set              |
| Station Name                  | (0008,1010)  | 3    | Always Set           |
| Institutional Department Name | (0008,1040)  | 3    | Not set              |
| Performing Physicians' Name   | (0008,1050)  | 3    | Set From User        |
| Operators' Name               | (0008,1070)  | 3    | Always Set           |
| Manufacturer's Model Name     | (0008,1090)  | 3    | Always set           |
| Device Serial Number          | (0018,1000)  | 3    | Not set              |
| Software Versions             | (0018,1020)  | 3    | Not set              |

#### 11.1.1.7. General Image Module

| Attribute Name      | Tag          | Type | Attribute Assignment |
|---------------------|--------------|------|----------------------|
| Instance Number     | (0020, 0013) | 2    | Always set           |
| Patient Orientation | (0020,0020)  | 2C   | Length=0             |
| Content Date        | (0008, 0023) | 2C   | Not set              |

*Kodak Point of Care* CR Systems, DICOM Conformance Statement

|                         |              |    |            |
|-------------------------|--------------|----|------------|
| Content Time            | (0008, 0033) | 2C | Not set    |
| Image Type              | (0008, 0008) | 3  | Not set    |
| Derivation Description  | (0008, 2111) | 3  | Not set    |
| Pixel Spacing           | (0028, 0030) | 3  | Always set |
| Lossy Image Compression | (0028, 2110) | 3  | Not set    |

#### 11.1.1.8. Image Pixel Module

| Attribute Name             | Tag          | Type | Attribute Assignment |
|----------------------------|--------------|------|----------------------|
| Samples per Pixel          | (0028, 0002) | 1    | 1                    |
| Photometric Interpretation | (0028, 0004) | 1    | MONOCHROME2          |
| Rows                       | (0028, 0010) | 1    | Always set           |
| Columns                    | (0028, 0011) | 1    | Always set           |
| Bits Allocated             | (0028, 0100) | 1    | Always set ( 16 )    |
| Bits Stored                | (0028, 0101) | 1    | Always set ( 12 )    |
| High Bit                   | (0028, 0102) | 1    | Always set ( 11 )    |
| Pixel Representation       | (0028, 0103) | 1    | Always set ( 0 )     |
| Pixel Data                 | (7FE0, 0010) | 1    | Always set           |
| Planar Configuration       | (0028, 0006) | 1C   | Not set              |
| Pixel Aspect Ratio         | (0028, 0034) | 1C   | Always set ( 1\1 )   |
| Smallest Image Pixel Value | (0028, 0106) | 3    | Always set           |
| Largest Image Pixel Value  | (0028, 0107) | 3    | Always set           |

#### 11.1.1.9. CR Image Module

| Attribute Name          | Tag          | Type | Attribute Assignment |
|-------------------------|--------------|------|----------------------|
| KVP                     | (0018, 0060) | 3    | Set in Grayling mode |
| Plate ID                | (0018,1004)  | 3    | Always set           |
| X-Ray Tube Current      | (0018, 1151) | 3    | Not set              |
| Exposure Time           | (0018,1150)  | 3    | Not set              |
| Exposure                | (0018,1152)  | 3    | Not set              |
| Exposure in uAs         | (0018,1153)  | 3    | Set in Grayling mode |
| Image Area Dose Product | (0018,115E)  | 3    | Set in Grayling mode |

*Kodak Point of Care CR Systems, DICOM Conformance Statement*

|                         |             |   |            |
|-------------------------|-------------|---|------------|
| Imager Pixel Spacing    | (0018,1164) | 3 | Always Set |
| Focal Spot              | (0018,1190) | 3 | Not set    |
| Relative X-Ray Exposure | (0018,1405) | 3 | Always set |
| Sensitivity             | (0018,6000) | 3 | Always set |

#### 11.1.1.10. Modality LUT Module

| Attribute Name        | Tag          | Type | Attribute Assignment |
|-----------------------|--------------|------|----------------------|
| Modality LUT Sequence | (0028, 3000) | 1C   | Not set              |
| >LUT Descriptor       | (0028,3002)  | 1C   | Not set              |
| >LUT Explanation      | (0028,3003)  | 3    | Not set              |
| >Modality LUT Type    | (0028,3004)  | 1C   | Not set              |
| >LUT Data             | (0028,3006)  | 1C   | Not set              |
| Rescale Intercept     | (0028,1052)  | 1C   | Always set ( 0 )     |
| Rescale Slope         | (0028,1053)  | 1C   | Always set ( 1 )     |
| Rescale Type          | (0028, 1054) | 1C   | Always US            |

#### 11.1.1.11. VOI LUT Module

| Attribute Name   | Tag         | Type | Attribute Assignment |
|------------------|-------------|------|----------------------|
| VOI LUT Sequence | (0028,3010) | 3    | Not set              |
| >LUT Descriptor  | (0028,3002) | 1C   | Not set              |
| >LUT Explanation | (0028,3003) | 1C   | Not set              |
| >LUT Data        | (0028,3006) | 1C   | Not set              |
| Window Center    | (0028,1050) | 1C   | Always set           |
| Window Width     | (0028,1051) | 1C   | Always set           |

#### 11.1.1.12. SOP Common Module

| Attribute Name         | Tag          | Type | Attribute Assignment                     |
|------------------------|--------------|------|------------------------------------------|
| SOP Class UID          | (0008, 0016) | 1    | Always set                               |
| SOP Instance UID       | (0008, 0018) | 1    | Always set                               |
| Specific Character Set | (0008, 0005) | 1C   | As in Support of Extended Character Sets |

